

spex

Shaping Better Lives



Tessellated
Shape System



Spex has taken the concept of tessellation, and applied this method to create contours for seating and positioning.

This system can be added to any back support from the Spex range for additional contouring behind the existing upholstery padding.

Upgrade Vigour, Manta or Spex back supports in order to maximize stability, positioning, and seating tolerance. Different densities are colour-coded for easy application.

Tessellated shape system can be used:

- To assist clients with mild to moderate postural needs when using a standard back support with flat padding
- In all situations where a supportive build-up foundation may be beneficial to assist positioning and comfort
- To improve sitting posture and enhance function (eg. Creating sacral/lumbar supports)

Easy & effective back support contouring.
Stack & configure each component!



**Blue Base
Building Blocks**

Firm shape is used to build a firm foundation for the contouring.



**White Intermediate
Building Blocks**

Medium shape is used to build up shapes where positioning is required.



**Charcoal
Overlay Blocks**

Softer shape is used for creating softer overlay over the previous blocks.



**Velcro
Fastening Strips**

These are used to smooth over the contours and hold it all together.

TSS Lite




Selection of key components from the TSS kit to enable fast contouring solution. Suitable for use on all Spex Back Supports, particularly for Zygo, Adapta and those with non-rectangular/flat surfaces.

1250-3955-000

TSS Elements

Comprehensive selection and quantity of TSS kit elements for a substantial contouring solution. Suitable for use on Spex Back Supports with a rectangular back surface, e.g. Manta, Vigour, Classic.

1250-3959-000

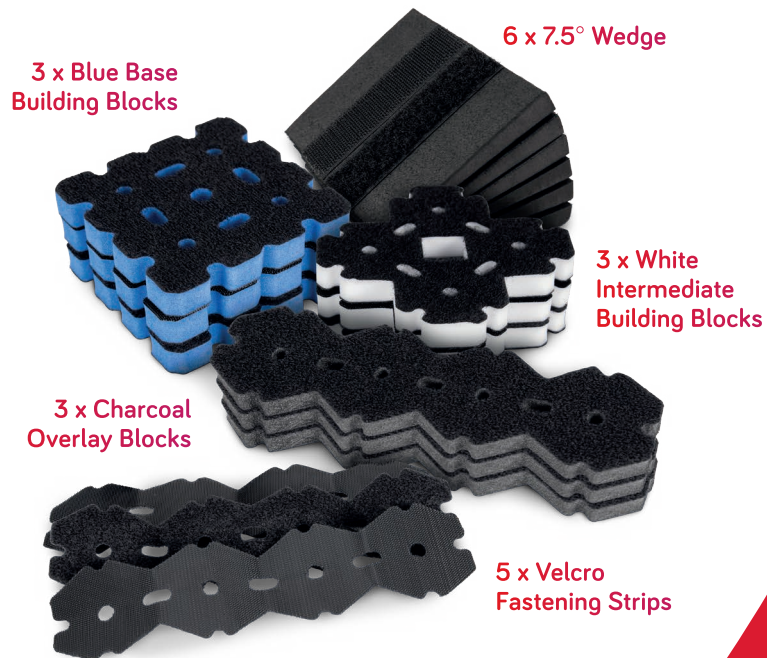


Tessellated Shape System

This system can be added to any back support from the Spex range for additional contouring behind the existing upholstery padding. Includes Velcro Backing Board.

1250-WWHH-000



Specifications

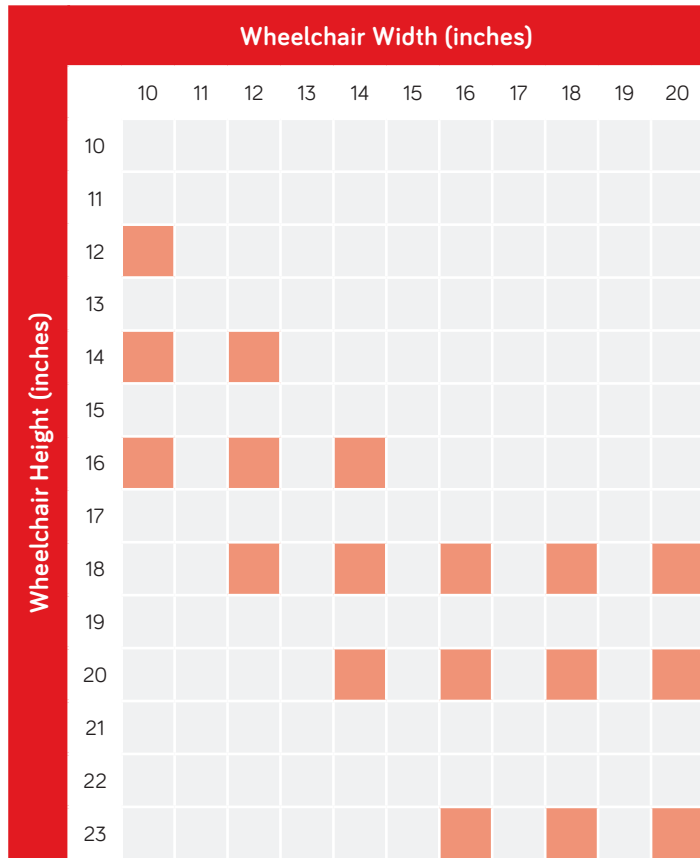

Product Weight

(16x16)

0.6kg

- Red squares are standard sizes.
- Non-standard or custom sizes are available on request.



Features & Benefits



Slots behind back support squab

Developed to be placed behind the back support squab (in front of the shell).



Endlessly configurable!



Shape to fit

Stack and configure the shapes according to the user's needs.



Varying foam

Positioning pack includes a number of different tessellations in varying foam.



Strong velcro backing

The different elements of the TSS are backed with a strong Velcro surface.



Cut to suit

The components of the TSS can be cut and modified to achieve the desired configuration, making it a versatile solution to easy and effective back support contouring.

The Spex logo is displayed in a white, lowercase, sans-serif font. It is positioned within a large, semi-circular graphic element that transitions from a dark red to a bright red color.

Manufactured by Spex Ltd
solutions@spexseating.com
spexseating.com

A white circular placeholder with the text "DISTRIBUTED BY:" inside. The rest of the circle is empty, intended for a distributor's name or logo.

DISTRIBUTED BY: